



Preservative free

Comfort Eye Drops

Singles - Carmellose Sodium 0.5% w/v

Composition:





Carboxymethylcellulose Sodium 5.0 mg, Sodium Chloride, Potassium chloride, Calcium chloride dihydrate, Magnesium chloride hexahydrate, Sodium lactate solution, Sodium hydroxide and/or hydrochloric acid to adjust pH, Water for injection.

Intended Use:

Ocufresh Comfort - Singles eye drops preservative-free is used to lubricate dry eyes.

Instructions for use:

- Wash and dry your hands thoroughly before use;
- Detach the single dose container from the strip (image 1) and open the single-dose container by twisting and pulling the tab (image 2);
- Hold the single-dose container into other hand, turn the container upside down and apply slight pressure to the single-dose container to release 1 or 2 drops of Ocufresh Comfort - Singles Eye Drops into the eye (image 3);
- Repeat the process for the other eye, if required;
- This product is not for repeat use, immediately discard the single-dose container after use (image 4);
- The product can be used as required or according to the advice of your doctor.

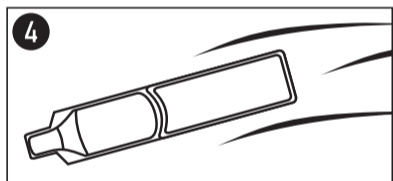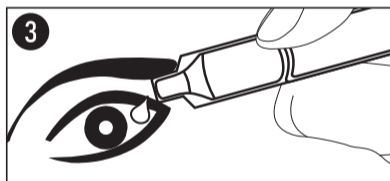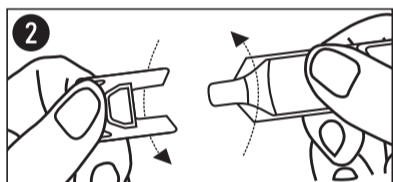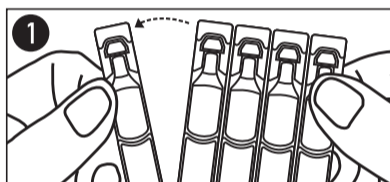

Warning and Precautions:

- This product is only for use in eyes;
- The product is intended for use only by the concerned patient;
- Do not use the product if the container is damaged when you open it for the first time;
- Do not use if the solution is cloudy or discoloured;
- Keep out of the sight and reach of children;
- Do not swallow;
- During the application of the eye drops, do not touch your eyes or any other surface with the dropper of the container to avoid any infections or damaging your eyes;
- Not for treatment of any pathological eye conditions;
- Do not re-sterilize the final packed product as it is not suitable for re-sterilization;
- The administration of other eye medicines at the same time as this product should be carried out under the supervision of your doctor who can evaluate possible interactions; there should be an interval of at least 15 minutes between the application of any other eye medicine and this product;
- Check the expiry date on the box before use; do not use the product after expiry date indicated on the package;
- After use, dispose of the product according to local regulations;
- Consult your doctor if symptoms persist.



Preservative free

Comfort Eye Drops

Singles - Carmellose Sodium 0.5% w/v

Possible side effects:

- As with other eye drops, this product can cause eye pain, redness or irritation of the eyes.
- If any of these effects persist or worsen or persist over 72 hours, promptly contact your doctor.

Package:

30 Single dose containers with a volume of 0.4ml.


Storage conditions:

- Store between 15°C and 25°C.
- Do not Freeze.
- Keep away from sunlight by placing the single-dose containers in the carton box.

NOT FOR INJECTION

MD

STERILE A

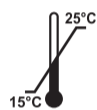

Store between
15°C and 25°C



Do not resterilise



Caution, see
instruction for use



Do not use if
package is damaged



Keep Dry



Do not reuse



Keep away from
Sunlight



Consult instructions
for use



Sentiss Pharma Pvt. Ltd.
Vill. Khara Nihla, Teh. Nalagarh,
Dist. Solan (HP)-174101, INDIA

Mfg. Lic. No.: MB/06/470



EC REP

OBELIS S.A
Boulevard General Wahis 53
1030 Brussels, BELGIUM
Tel: +(32) 2.732.59.54
Fax: +(32) 2.732.60.03
E-Mail: mail@obelis.net



BLUMONT

Blumont Healthcare Limited.

23 Moortown Close, Grantham,
NG31 9GG, United Kingdom

XXX/XXXX/XX