150 mm

200 mm

# OCAL® EUPHRASIA PLUS 100ml

Eye Wash | Oogbad | Bain Oculaire | Augenbad

with **Aloe Vera**

## EN - EYE WASH

Eye wash solution with Sodium hyaluronate, Euphrasia extract (Euphrasia officinalis) and Aloe Vera gel. For moisturising and cleaning dry, tired, red, gritty or irritated eyes and eyelids.

### Indications

- Relieves dry, tired, red, gritty or irritated eyes.
- Relieves environmental ocular stress due to excessive exposure to wind, dust, air conditioning, dry and polluted air, closed and smoky rooms, or prolonged contact with sea or swimming pool water.
- Relieves visual stress due to prolonged computer use, unfavourable lighting conditions or night driving.
- For rinsing dust, pollen and other foreign bodies from the eye.
- For rinsing eye cream and eye ointment residues from the eye.
- Provides relief in case of prolonged use of contact lenses.
- Compatible with homeopathic treatments.
- For children over 3 years of age.

### Instructions for Use

#### Use with eye bath

- Before each application, rinse the eye bath well with the solution.
- Fill the eye bath with the solution.
- Rest the eye bath on your eye.
- Slightly tilt your head keeping your eye open to allow contact with the solution for a few seconds.
- If necessary, repeat the operation several times, replacing the solution each time.
- Rinse the eye bath with water after each use.
- Dispose according to local laws.

#### Use without eye bath

- Irrigate your eye with abundant solution being careful never to let the tip of the bottle touch the eye.
- Wait a few seconds and repeat the operation a few times.
- For a more effective result, you can repeat the treatment several times a day.
- Dispose according to local laws.

### Warnings

- In case of eye irritation, stop using the product and contact a specialist.
- Do not use in case of allergy or hypersensitivity to any of the product components.
- Store in a dry place between 5 and 35°C.
- Keep out of the reach of children
- Do not swallow.
- Do not let the tip of the bottle touch your eye or eyelid.
- Do not mix the solution with other fluids.
- Do not use the product while wearing contact lenses.
- Always wash the eye bath before using it with the solution.
- Do not use the product after expiry date.
- Use within 90 days of first opening
- Sterile until opened.
- Do not use the product if the bottle appears damaged.
- Do not reuse the solution.
- Close the bottle firmly after each use.

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The carton, container and instruction leaflet are recyclable.

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Blumont Ota Trading Ltd, MT c/o ID Phar,