OCAL[®] Eye Bath | Oogbad | Bain Oculaire | Augenbad

100ml

EN - OCAL[®] EYE BATH

EYE WASH

Eye solution for the hydration and hygiene of tired, sensitive and red eyes and of periorcular area and eyelids.

Indications

- Ideal in case of red, tired and sensitive eyes;
- provides relief in case of dry eyes and foreign body sensation;
- provides relief in case of environmental ocular stress due to excessive exposure to wind, dust, air conditioning, dry and polluted air, closed and smoky rooms;
- provides relief in case of visual stress due to prolonged computer use, unfavourable lighting conditions or night driving;
- ideal for irrigation and eye washing for children over 3 years of age;
- ideal to rinse the eye from dust, pollen and other foreign bodies;
- ideal in case of prolonged contact with sea or swimming pool water;
- ideal to rinse the eye from residues of creams and ointments;
- provides relief in case of prolonged use of contact lenses;
- use the eyewash solution with the supplied eye-cup or directly from the bottle;
- compatible with homeopathic treatments.

Instructions for use

Use with eye-cup

- Before each application, rinse the eye-cup well with the solution;
- fill the eye-cup with the solution;
- rest the eye-cup on your eye;
- slightly tilt your head keeping your eye open to allow contact with the solution for a few seconds;
- if necessary, repeat the operation several times, replacing the solution each time;
- rinse the eye-cup with water after each use;
- dispose according to local laws.

Use without eye-cup

- Irrigate your eye with abundant solution being careful never to let the tip of the bottle come into contact with it. Wait a few seconds and repeat the operation a few times. For a more effective result, you can repeat the treatment several times a day.
- Dispose according to local laws.

Warnings

- In case of eye irritation, stop using of the product and contact a specialist;
- do not use in case of allergy or hypersensitivity to any of the product components;
- keep away from heat sources, store in a dry place at temperatures between 5 and 35°C;
- keep out of reach of children;
- do not swallow;
- do not let the tip of the bottle touch your eye or eyelid;
- do not mix the solution with other fluids;
- do not use the product while wearing contact lenses;
- always wash the eye-cup before using it with the solution;
- do not use the product after expiry date;
- use within 90 days of first opening;
- sterile until opened;
- do not use the product if the bottle appears damaged;
- do not reuse the solution;
- close the bottle firmly after each use.

Composition

Phosphate buffer, Disodium edetate, PHMB isotonic solution buffered to pH 7.2.

Does not contain

Benzalkonium chloride, Chlorhexidine, Parabens, Phenoxyethanol, Thimerosal, and components from animal origin.

 Carton, container and Instructions leaflet are recyclable

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Blumont Ofta Trading Ltd,