150mm

200mm

## Contact Lens Solution with Sodium Hyaluronate and Aloe Vera 100ml/360ml

All-in-one

### EN Indications

Disinfection, storage, cleaning, hydration, lubrication, rinsing and removing protein and lipidic residues from the contact lenses surface. Particularly indicated for rigid contact lenses.

**Read the instruction for the use and the information for safety before use**

#### Mode of use

- Wash and dry your hands carefully;
- fill both section of the lens case with the solution, making sure that it is empty and clean first, and place each contact lens in the appropriate compartment of the lens case;
- close the lens case firmly and leave the contact lenses in the solution for at least 6 hours, better overnight;
- shake the lens case gently before opening;
- wash and dry your hands carefully before removing the contact lenses from their case;
- place the lens in the palm of your hand and pour the solution from the bottle on to the lens and gently rub each of the surfaces;
- rinse the contact lenses with fresh solution before use;
- empty the lens cases after each use, rinse it with the solution and leave to air dry;
- repeat the procedure if the lens remain in the lens case for more than 7 days;
- dispose the product and the lens case according to local laws.

#### Contraindications

Do not use in case of allergy or hypersensitivity to any of the product components.

#### Limitations, Precautions and Warnings

- In prolonged burning sensation or in case of eye irritation, remove contact lenses, stop using the product and contact a specialist;
- keep away from heat sources, sunlight and store in a dry place, at temperature between 5 and 35°C;
- do not freeze;
- keep out of the reach of children;
- do not swallow. In case of ingestion contact a doctor;
- do not mix with other solutions;
- do not use after the expiry date;
- use within 90 days of first opening;
- sterile until opened;
- do not use the product if the bottle or the lens case are damaged or not perfectly closed at first use;
- do not reuse the solution;
- do not touch the opening of the bottle with fingers or other surfaces;
- close the bottle firmly after each use;
- during disinfection always keep your lens case in an upright position;
- do not share the lens case with other user;
- clean regularly and change frequently the lens case as recommended by your specialist;
- in case of serious incident notify the event to the manufacturer and to the local competent authority;
- keep the box and the leaflet.

#### Composition

Polyhexamethylene biguanide (PHMB) 0.000432%, purified water, sodium citrate, pluronic, sodium chloride, disodium edetate, Aloe vera gel, Hydroxypropyl methylcellulose (HPMC), sodium hydroxide, citric acid, Polyvinylpyrrolidone (PVP), sodium hyaluronate.

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